



NQSBG

**General
bacteriology**

Description

Sent sample: lyophilized pellet of a single microorganism strain

Instruction for use:

Method 1:

Place the pellet on a plate of proper culture medium, crawl and incubate

Method 2:

Transfer the pellet into a tube containing 2-3 ml of proper liquid culture medium and incubate. The sample instead of liquid culture medium can be dissolved with 100 µl of sterile water.

Test the sample as it is a patient sample.

Storage:

Store the sample at 2-8 °C.

Sample reconstitution

Before reconstitution of the samples, bring it to room temperature.

Precautions:

The sample has been tested according to the specifications and met the quality standards.

Analytes:

Identification and susceptibility testing.

ORIGINE OF THE SAMPLE: SEE LABEL