



**NQSMC**

**General  
micology**

**Description**

Sent sample: lyophilized pellet of a single microorganism strain

**Instruction for use:**

**Method 1:**

Place the pellet on a plate of proper culture medium, crawl and incubate

**Method 2:**

Transfer the pellet into a tube containing 2-3 ml of proper liquid culture medium and incubate. The sample instead of liquid culture medium can be dissolved with 100 µl of sterile water.

Test the sample as it is a patient sample.

**Storage:**

Store the sample at 2-8 °C.

**Sample reconstitution**

Before reconstitution of the samples, bring it to room temperature.

**Precautions:**

The sample has been tested according to the specifications and met the quality standards.

**Analytes:**

Identification and susceptibility testing.

**ORIGINE OF THE SAMPLE: SEE LABEL**