



NQSHG

HbA1c

Description:

Sent sample: 1 vial **0,5 ml, lyophilized.**

Instruction for use:

Reconstitute the sample with **0,5 ml distilled water.**

Storage:

Store the sample at **2-8 °C.**

Sample reconstitution

Before reconstitution of the sample, bring it to room temperature.

Open the vial carefully, and reconstitute the sample with the exact quantity of distilled water (**0,5 ml**).

After 5-10 homogenize the sample by stirring it gently. Leave the sample **30 minute** until the lyophilized material is totally dissolved.

Analyze the control material **as a patient sample**, taking into consideration the instruction of use of the instrument and reagents.

After reconstitution the sample is stable **4 hours** at **25°C.**

Precautions:

The serum has been tested for Human Immunodeficiency Virus (HIV) antibody, Hepatitis B Surface Antigen (HbsAg), and Hepatitis C Virus (HCV) antibody and found to be NON-REACTIVE. However this material should be handled carefully.

Parameters:

HbA1c