



NQSHP

**Helicobacter
pylori antigen**

Description:

Sent sample: 1 vial of **1 ml synthetic fecal sample**

Instruction for use:

Reconstitute the sample with **0,25 ml distilled water**.

Storage:

Store the sample at **4-30 °C**.

Sample reconstitution:

Open the vial carefully, and reconstitute the sample with the exact quantity of distilled water (**0,25 ml**). Leave the sample **30 minute** until the lyophilized material is totally dissolved. During the reconstitution rotate and stir gently the vial for homogenization.

Analyze the control material **as a patient serum**.

Precautions:

The serum has been tested for Human Immunodeficiency Virus (HIV) antibody, Hepatitis B Surface Antigen (HbsAg), and Hepatitis C Virus (HCV) antibody and found to be NON-REACTIVE. However this material should be handled carefully.

Parameters:

Helicobacter pylori Antigen